

Customizing Your Body Analysis Monitor

This scale can be customized for up to 4 users. While the customization process is not difficult, it may be easiest to read all the instructions first to familiarize yourself with them. Then keep the instruction manual handy as you go through the process step by step.

Buttons: SET, Up and Down arrows

1. To set your user data

Gently push on the scale with your foot. "000" will appear, then "0.0" will display.

2. Setting the user number

Press the SET button. "P-01" will appear and flash. Use the Up or Down arrow to select a user number (P-01 to P-04). Press SET to confirm.



3. Setting the height

Next, the inch or cm symbol will light up (if switch is set to lb, default is height in inches; if switch is set to kg, default is height in cm). Press the Up or Down arrow to adjust the height. Pressing and holding either the Up or the Down arrow will advance numbers quickly. Press SET to confirm the height.



4. Setting the age

The default age, 25, will appear and flash. Press the Up or Down arrow to adjust the age and press SET to confirm.



5. Setting the gender

The gender icon will flash. Press the Up or Down arrow to choose male or female gender. Press SET to confirm.



6. Displaying and storing data

Once gender is set, data storage is complete. Scale will display your stored data (user number, height, age and gender) for a few seconds.

7. The scale is ready for use.

"0.0" will show on the display. You may press the SET button to revise the user data or set up data for different user, or you can now use the scale to take your body analysis readings. Or, you can let the display turn off automatically after a few seconds and use the scale later.

Body Analysis

1. Operate the scale on a secure, flat floor. Gently press the scale with your foot until you see "000" flashing on the display. Take your foot off the scale immediately, and wait until "0.0" appears.



2. Press the Up or Down arrow to select your user number and recall the stored data. Then "0.0" will appear again.

3. Step on the scale barefoot with your feet aligned on the electrodes (metal strips) on both sides of the platform.

NOTE: You must be barefoot in order for the scale to read your body analysis measurements.

4. Your current weight will be displayed for a few seconds.



5. Next, the display will show a running 0 to show it is entering the body analysis mode.



6. Then the display will show your body fat.



7. Then it will show your body water percentage.



8. Next, the display will show muscle mass.



9. Then it will show your bone mass.



10. And finally, the display will show your BMI value.



11. The LCD will display your current weight, body fat, body water, muscle mass, bone mass and BMI again, and then turn off automatically.

NOTE: If an error occurs, or weight is over maximum capacity, the scale will not function properly and "Err" will appear.

Scale will show "Err" if there is improper foot contact in the body analysis mode, for instance, if you stand on the scale with socks or shoes on. Hard or dry skin can also prevent you from making proper contact with electrodes on the surface of the scale. Try applying a small amount of moisturizer to your feet before using the scale.



Manual App Tracking

Your scale will work with the free WW Scales by Conair® app to give you an easy-to-use way of tracking your measurements. To download the app, go to the Apple® App Store or Google Play app store and type WW scales by Conair into the search box. Once you have downloaded the app to your device, it will walk you through the registration process. Note that if you are a WW member and would like to have your weight data sync automatically to the WW member app, you will need to register with the same user name and password you are currently using for the WW member app. Once you are registered, you can manually add data into the app tracker by tapping the "+" sign on the dashboard: you can choose to add data every time you weigh yourself, or just add data on certain days. You can choose to add just one measurement (for instance, weight) for tracking, or you can add any or all of the body analysis measurements, such as body fat, body water, bone mass, muscle mass and BMI. The app gives you an easy, fun way to keep track of the progress you are making.



Supplier's Declaration of Conformity

Brand: WW by CONAIR

Model No.: WW714ZF

Description: Digital Body Analysis Scale

Responsible Party: Conair LLC, 1 Cummings Point Road, Stamford CT 06902 (203) 351-9000

Standards: FCC Part 15 Subpart B, ANSI C63.4-2014

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

Date of Issue: June 18, 2021

FCC Markings:

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Caution: Any changes or modifications not expressly approved by Conair LLC, the manufacturer of this product and the party responsible for compliance, could void the user's authority to operate the equipment.

This scale is not a toy.

LIMITED TEN-YEAR WARRANTY (U.S. & CANADA ONLY)

Conair will repair or replace, at our option, your appliance free of charge for 120 months from the date of purchase if the appliance is defective in workmanship or materials.

To obtain service under this warranty, return the defective product to the service center listed, together with your sales receipt. California residents need only provide proof of purchase and should call 1-800-3-CONAIR for shipping instructions. In the absence of a receipt, the warranty period shall be 120 months from the date of manufacture.

ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL BE LIMITED IN DURATION TO THE 120-MONTH DURATION OF THIS WRITTEN LIMITED WARRANTY. Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL CONAIR BE LIABLE FOR ANY SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY, EXPRESS OR IMPLIED, WHATSOEVER. Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state.

Please register this product at www.conair.com/registration

Spanish translation available online at www.conairscales.com



five body analysis results



manually track on free app



2.6" backlit display



user memory

Instructions for use.
To get the greatest benefit from your purchase, please read this entire brochure before using your scale.

WW714ZF

