USER MAINTENANCE INSTRUCTIONS

CARING FOR YOUR TRUE GLOW® LIGHT THERAPY DEVICE

MAINTENANCE:

1. This is a medical device; it is important to keep it clean.
2. There are no user serviceable parts in this device.
3. The device includes only an On/Off button: All settings are set at the factory.
4. Use only supplied USB cord and adapter.

CLEANING:


1. Turn device off and unplug from any power source.
2. Wipe the tops of LEDs with alcohol wipes.
3. Clean handle with damp cloth or rubbing alcohol.
4. Allow to dry completely prior to use.

TROUBLE-SHOOTING GUIDE

SYSTEM NOT TURNING ON.	
	1. Is power cord pushed firmly into outlet?
	2. Are batteries fully charged?
	3. Is USB power source turned on?
	4. Is power cord pushed firmly into handle?
	5. Is On/Off button turned on?
	6. If device still does not turn on, call Customer Service.

**WARNING:
IF YOU THINK THE
POWER IS OFF
WHEN THE SWITCH
IS OFF,
YOU'RE WRONG.**

**KEEP AWAY
FROM WATER**

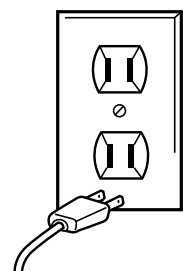Everyone knows that electricity and water are a dangerous combination. But did you know that an electrical appliance is still electrically live even if the switch is off? If the plug is in, the power is on. So when you are not using your appliances, keep them unplugged.

**ALWAYS UNPLUG SMALL
APPLIANCES**

DECLARATION OF CONFORMITY
True Glow® for Wrinkles/Acne conforms to the International Medical Device EMC/EMI Standard Regulations: EN60601-1-2.

CAUTION: Any changes or modifications not expressly approved by Conair Corporation, the manufacturer of this product and the party responsible for compliance, could void the user's authority to operate the equipment.

FCC MARKINGS

WARNING: Changes or modifications to this unit not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

NOTE: This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

LIMITED ONE YEAR WARRANTY (U.S. AND CANADA ONLY)

Conair will repair or replace (at our option) your unit free of charge for 12 months from the date of purchase if the appliance is defective in workmanship or materials.

To obtain service under this warranty, return the defective product to the service center listed on back that is nearest you, together with your purchase receipt and \$3.00 for postage and handling. California residents need only provide proof of purchase and should call 1-800-3-CONAIR for shipping instructions. In the absence of a purchase receipt, the warranty period shall be 12 months from the date of manufacture.

ANY IMPLIED WARRANTIES, OBLIGATIONS, OR LIABILITIES, INCLUDING BUT NOT LIMITED TO THE IMPLIED WARRANTY OF MERCHANTABILITY AND FITNESS FOR A PARTICULAR PURPOSE, SHALL BE LIMITED IN DURATION TO THE 12-MONTH DURATION OF THIS WRITTEN, LIMITED WARRANTY.

Some states do not allow limitations on how long an implied warranty lasts, so the above limitations may not apply to you.

IN NO EVENT SHALL CONAIR BE LIABLE FOR ANY SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES FOR BREACH OF THIS OR ANY OTHER WARRANTY,

EXPRESS OR IMPLIED, WHATSOEVER.

Some states do not allow the exclusion or limitation of special, incidental, or consequential damages, so the above limitation may not apply to you.

This warranty gives you specific legal rights, and you may also have other rights, which vary from state to state.

Please register this product at www.conair.com/registration

SERVICE CENTERS CONAIR CORPORATION

Service Department

150 Milford Road

East Windsor, NJ 08520

CONAIR CORPORATION

Service Department

7475 North Glen Harbor Blvd.

Glendale, AZ 85307

Visit us on the web at: www.conair.com

©2016 Conair Corporation
East Windsor, NJ 08520
Glendale, AZ 85307

Made in China

TRA1 Models
16PS019317

IB-14031B

All TRA1 Models

renew · refresh · revive
true glow

Light Therapy Solution
for professional results

CONAIR

IMPORTANT SAFETY INSTRUCTIONS

When using electrical appliances, especially when children are present, basic safety precautions should always be taken, including the following:

READ ALL INSTRUCTIONS BEFORE USING THE LIGHT THERAPY DEVICE

1. Close supervision is required when this product is used on, by or near children or people with certain disabilities.
2. Use product only for its intended use as described in this instruction booklet. Do not use attachments not recommended by the manufacturer.
3. Keep the cord away from heated surfaces.
4. Never use while sleepy or drowsy.
5. Do not use outdoors or where oxygen is being administered.
6. Never operate if the product is damaged.
7. Do not operate the adapter if it has a damaged cord or plug, if it is not working properly, if it has been dropped or damaged or dropped into water.
8. Use only with Conair-supplied power adapter.
9. Do not operate device where it could come into contact with water.
10. Do not use the device if power supply has fallen into water or has been damaged in any way
11. Stop using device immediately if you think it is defective. Please contact 1-800-3-CONAIR immediately.

12. Do not bend or abuse the wire of the power supply as this can cause electrical shock and problems.
13. Do not attempt to take device apart. The device is not serviceable by the user. This will void the warranty.

PRECAUTIONS

1. Always unplug power supply from wall outlet when not in use.
2. Do not store device in sun or any location with high temperatures as this may cause damage to plastic casing.
3. Use only the supplied power supply.

CAUTIONS

1. Do not use this device on infants and young children without physician approval.
2. Do not shine the device lights directly into the eyes.
3. Discontinue use if device overheats or becomes uncomfortably hot to your skin.
4. Do not use this device if your skin is light sensitive or you are currently using medication that may cause skin sensitivity to light.
5. Contact your doctor prior to using if you:
 - Are pregnant or suspect you may be pregnant
 - Are taking drugs known to cause light sensitivity
 - Have any suspicious or cancerous lesions
6. Contact your doctor if you have any medical questions regarding use of this device.
7. Contact your doctor if you suspect that you have had a negative reaction from using the device.

The True Glow® Light Therapy Device handle has no serviceable parts. If the power cord is damaged, it must be unplugged and replaced. Call 1-800-3-CONAIR for instructions.

LIGHT SENSITIVITY AND CAUTIONS SPECIAL CAUTIONS FOR DELICATE AREAS

Test yourself for light sensitivity prior to using on your face:

1. Power on your device.
2. Place device light head on or within ¼" of the skin on your forearm for 3 minutes.
3. After 3 minutes, check the area. If area has turned red and it lasts for more than 2 hours, your skin is light sensitive.
4. Discontinue use and contact Customer Service.

TRUE GLOW® LIGHT

THERAPY DEVICE FEATURES AVAILABLE POWER SOURCES FOR POWERING THE LIGHT THERAPY DEVICE

BATTERIES

Unit may be powered by 3 AA batteries (not included). Rechargeable AA batteries are acceptable. Replace batteries when light begins to dim.

USB POWER SOURCE

Device may be powered by any USB power source, including laptop, car charger, external USB power supply, etc.

110V WALL OUTLET

Device may be powered by standard US 110V wall outlet by connecting USB power cord to 110V adapter (included), plugged into wall outlet (adapter does not function as a charging unit).

TRUE GLOW® LIGHT THERAPY FOR ACNE

This is a handheld light emitting diode (LED) device that emits energy for use in dermatology for the treatment of mild to moderate inflammatory acne. The device delivers natural blue light energy in the 415nm spectrum that is proven to destroy acne-causing bacteria within the skin. The device treatment time is controlled by the operator. There are no user settings or adjustments.

OPERATING INSTRUCTIONS

NOTE: Do not apply treatment close to the eyes to avoid risk of injury.

1. Wash and clean face prior to use.
2. Remove the protective cover from the lights.
3. Select the power source you desire and ensure device has power by pressing On/Off button on back of device.
4. Fit user relaxation goggles for comfort.
5. Place the device head within ¼" of the skin on the treatment area.
6. Leave the light on the treatment area for a minimum of 3 minutes. Do not move or wave the device across the treatment area.
7. Use a clock or timer to time your treatments.
8. Move device to the next treatment area and repeat.
9. Press the power button to turn the unit off.
10. Cover the lights with the plastic cover to protect lights from potential damage during storage.
11. Use the device daily for best results.
12. Device is equipped with an Auto Shut off function after 20 minutes in the case that device was accidentally powered on after use.

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